

Peelable Casting Tape



**Why Cut
When You Can Peel?**

Just Peel

No **Electric Saw** Needed



Features

- Good Strength and Rigidity
- Light Weight and Durable
- Excellent Conformability and Breathable
- Soft Edges and Smooth finish
- Easy and Convenient Removal
- Less To No Dust
- X-ray Translucent

U-Kast® Peelable Polyester Casting Tape is a revolutionary and innovative immobilizer for fresh and unstable bone fractures, joints injury and sprain. The stretchable property allows easy application without any tucks and folds, curing time of 3-4 minutes with weight bearing achieved in about 20 minutes.

Safe and easy option for removing tubular bandages. U-Kast® Peelable Polyester Casting Tape can be easily removed by hand, no electric saw needed. No dust is generated during removal.

Operating Instructions

- 1** Padding on the affected area
- 2** Wear gloves
- 3** Open the package and take out the tape
- 4** Soak the cast tape in water for about 5 seconds
- 5** Wrap the tape around the affected area
- 6** Smooth and wait for tape curing



Removal Method

U-Kast® Peelable Casting Tape can be removed easily by hand. Wear cotton gloves before operation. Peel off the bandage slowly with force, use tools such as a small spatula to help peel off the difficult parts.

Product Variant


Size	Package	Applicable Parts
2"x 4y / 5 x 360cm	10rl x 10box/ctn	Hand, Forearm
3"x 4y / 7.5 x 360cm	10rl x 10box/ctn	Forearm, Foot
4"x 4y / 10 x 360cm	10rl x 10box/ctn	Upper arm, Crus
5"x 4y / 12.5 x 360cm	10rl x 10box/ctn	Thigh, Torso
6"x 4y / 15 x 360cm	10rl x 10box/ctn	Thigh, Torso

Multi-color Options
(color is customizable)



Indications

Fresh and unstable bone fractures, joints injury and sprain

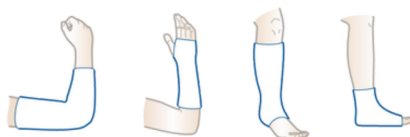

Manufactured by:
SHANGHAI YOUJIA MEDICAL SUPPLY CO., LTD.

BLDG 2B3-4, No.499, Fengcun Road, Fengxian District, Shanghai, China

Tel:+86 21 57569919/+86 21 57569920

Mob: +86 186 2111 8038

Email:export@ukast.cn



www.ukast.com