

# Self-Adhesive Ice Wrap

## Instant Cooling

Long Lasting Effect for  
Pain and Swelling

No Refrigeration Needed  
Reusable



U-Kast® Self-Adhesive Ice Wrap combines the benefits of cooling and compression therapy into a single, easy-to-use product.

It helps you heal more quickly and effectively from sprains and strains. It's also an ideal post-exercise aid for soothing sore or tired muscles.

## Features

- Effective for pain and swelling from sprains and strains
- Cooling and compression 2 in 1 therapy
- Promotes faster injury recovery
- Elastic bandage provides immediate cooling and adequate pressure
- Offers continuous cooling for up to 6 hours
- No need for refrigeration, can be used anytime
- Skin-friendly and latex-free
- Natural and odorless, no borneol, alcohol, or irritating ingredients
- Self-adhering with no odor or mess
- Reusable for multiple applications



## Operating Instructions

<p>Open the package and take out the tape</p>	<p>Gently wrap around the affected area</p>	<p>Adjust the tension of the tape while winding and wrapping, swelling parts can be adequately compressed</p>
<p>Massage the end of the tape gently, till the tape is fully adhered.</p>	<p>Elevate the affected body area and let it rest.</p>	<p>For reuse, immerse the whole tape into a jar with tap water for half hour, drain off excess water before reapplying, do not squeeze the tape.</p>

### WARNING:

1. Do not apply on open wounds.
2. To avoid circulation problems, do not wrap the tape too tight.
3. If irritation occurs or pain persists, discontinue use and seek medical advice or consult a medical professional.
4. If discomfort or adverse symptoms such as redness and itching of the skin arise, stop using the tape, or seek medical professional for advice.
5. Avoid contact with eyes or mouth. Do not swallow.
6. Store out of reach of children.



## Indications

- Muscle sprains and strains
- Post-exercise muscle fatigue



## Product Variant

Size (cm)	Size (cm)
5 x 100cm	5 x 300cm
7.5 x 100cm	7.5 x 300cm
10 x 100cm	10 x 300cm

### Manufactured by:

**SHANGHAI YOUJIA MEDICAL SUPPLY CO., LTD.**

BLDG 2B3-4, No.499, Fengcun Road, Fengxian District, Shanghai, China

Tel: +86 21 57569919/+86 21 57569920

Mob: +86 186 2111 8038

Email: export@ukast.cn

