

Elite POP Bandage



Creamy and Easy Moulding Casts for Traditional Application



Features

- Highly creamy texture
- Low plaster loss
- Fast set time
- Ease of moulding
- Low exotherm
- Strong and durable
- Smooth and appealing finish



U-Kast® Elite POP Bandage is developed by Shanghai YouJia Medical Supply Co., Ltd. after more than 20 years of continuous upgrading and improvement.

U-Kast® Elite POP bandage consists of a high quality cotton gauze coated with formulated plaster of Paris powder. This product features a creamy plaster for a smoother, stronger cast. It is designed for rapid and even wetting upon immersion. It forms a hard and rigid cast within 2-3 minutes of setting time while early weight bearing is achieved in 30 minutes. It is applied as a full cylindrical cast and/or back slab/splint.

Product Variant

Size	Package	Size	Package
5 x 270 cm	192 rolls/carton	12.5 x 270 cm	72 rolls/carton
7.5 x 270 cm	144 rolls/carton	15 x 270 cm	72 rolls/carton
10 x 270 cm	96 rolls/carton	20 x 270 cm	48 rolls/carton

Operating Instructions

The diagram illustrates the six steps for using the bandage:

- 1** Put a plaster liner on the affected area
- 2** The doctor put on his gloves.
- 3** Open the package and take out the bandage.
- 4** Soak the bandage in water for about 5 seconds
- 5** Wrap the bandage around the affected area.
- 6** Smooth and wait for the bandage to solidify.

Indications

Immobilization for acute and secondary fractures, sprains, corrective procedure of other body parts normally affecting upper and lower limbs.

